


# BIOLOGICAL MONITORING

V-PRO™ LOW TEMPERATURE STERILIZERS

**Materials:**



Self Contained Biological Indicator

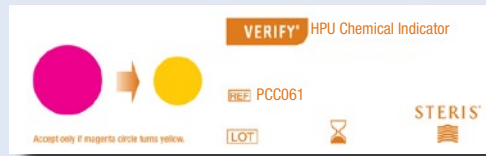

Chemical Indicator Strip



Low Temperature Sterilization Pouch

**NOTE: Ensure that the biological indicator and chemical indicator are within expiration.**


**1** Place one (1) biological and chemical indicator in the pouch. Seal the pouch.




NOTE: Cap must freely turn




**2** Place pouch at the center of the top shelf if both shelves are used or near the door if one shelf is used.

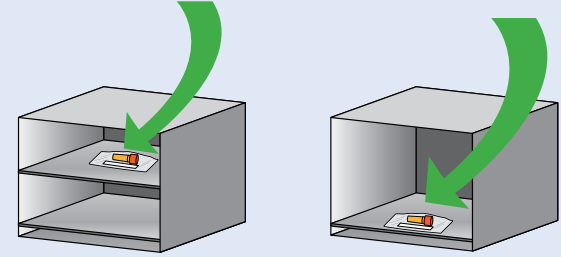


**3** Following sterilization, remove pouch and interpret chemical indicator.

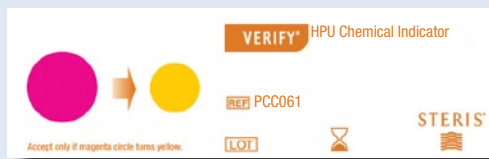

UNEXPOSED

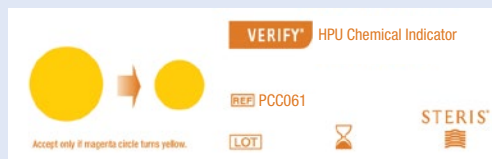

PASS

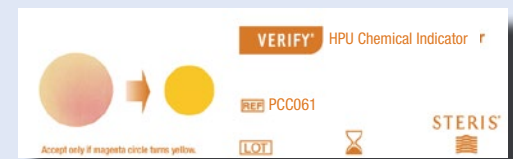

FAIL

**4** Seal, crush and incubate at 55-60°C for 24 hours.



**5** Interpret and record results.



Yellow

Positive Growth



Turbidity



Negative (No Growth)