



VERIFY® Bowie-Dick Test Card

VERIFY Bowie-Dick Test Card is the smallest and most simple pre-assembled Bowie-Dick test pack product available.

Description

VERIFY Bowie-Dick Test Card is the smallest and most simple pre-assembled Bowie-Dick test pack product available. The Card is the first test that generates no waste at all. The Card is the test. The Card provides a convenient system for testing prevacuum sterilizers.

The VERIFY Bowie Dick Test Card consists of a card printed on one side with bars of chemical indicator ink. On the opposite side of the card, space is provided for critical cycle information to be stored.

Upon completion of the Bowie-Dick test cycle, you will have immediate validation of the air removal process. The Card is easy to use. In an otherwise empty chamber, place the VERIFY Bowie-Dick Test Card, with chemical indicator bars facing up, in a test card holder on the lowest shelf over the sterilizer drain. During the vacuum phase of the cycle, air is removed from this uniquely designed product. Following air removal, the chemical bars uniformly darken with exposure to steam. If air is trapped in the card during the exposure phase of the cycle, all the bars will not show even color change. It's that simple.

Features

- Provides a convenient package for testing of prevacuum cycle steam sterilizers
- The Bowie Dick Test card is preassembled
- The test card can be retained as a permanent record
- Lot number and expiration date on each test box and storage box

Benefits

- Packaged in plastic storage boxes, the box provides convenient, permanent storage for approximately 2 months of processed cards
- A non uniform color change provides an immediate indication of inadequate air removal from the sterilizer chamber during the cycle
- Consistent quality is maintained. No material or labor resources are required to prepare the test card at the point of use
- Permanent documentation of the cycle parameters, date and test results
- Traceability and recognition of expired packs at time of use

PRODUCT NUMBER	DESCRIPTION	QUANTITY
S3095	VERIFY Bowie Dick Test Card	Box (31 tests)
S3096	VERIFY Bowie Dick Test Card Holder	Each

Literature

BROCHURE

Document #	Document Title
	VERIFY EQUIPMENT CONTROL PRODUCTS

SAMPLE PROCEDURES

Document #	Document Title
	SAMPLE PROCEDURES - STERILITY ASSURANCE PRODUCTS - S3095

SDS

Document #	Document Title
	VERIFY BOWIE DICK TEST CARD
	VERIFY BOWIE DICK TEST CARD - UNITED KINGDOM

SELL SHEET

Document #	Document Title
	BOWIE DICK TEST CARD

TECH DATA SHEETS

Document #	Document Title
	VERIFY S3095 BOWIE-DICK TEST CARD, TEST CARD FOR MULTIPULSE PREVACUUM STEAM STERILIZERS

TECHNICAL BULLETINS

Document #	Document Title
	VERIFY BOWIE-DICK TEST CARD

TECHNICAL TIPS

Document #	Document Title
	TECHNICAL TIP 5508 - TROUBLESHOOTING DART OR VERIFY BOWIE-DICK TEST CARD RESULTS
	UNDERSTANDING THE COMPLEMENTARY ROLES OF AIR REMOVAL (BOWIE-DICK), LEAK AND BIOLOGICAL TESTING IN STEAM STERILIZATION

Related Products

Other Sterility Assurance and Monitoring Products



Biological Indicators

- For routine monitoring, product testing and qualification of steam, EO, and vaporized hydrogen peroxide sterilization processes



Chemical Indicators

- Internal chemical indicator strips and challenge packs capture sterilization or disinfection failures through immediate results



Daily Air Removal Test/Bowie Dick Test

- Evaluate the performance of prevacuum sterilizers by confirming adequate air removal from the sterilizer chamber



Reusable Test Packs

- Designed to challenge vacuum assisted steam sterilizers, these reusable test packs offer environmental and budgetary benefits



Record Keeping and Labeling

- Envelopes, forms, record cards and labels to quickly document common sterilization, high level disinfection and